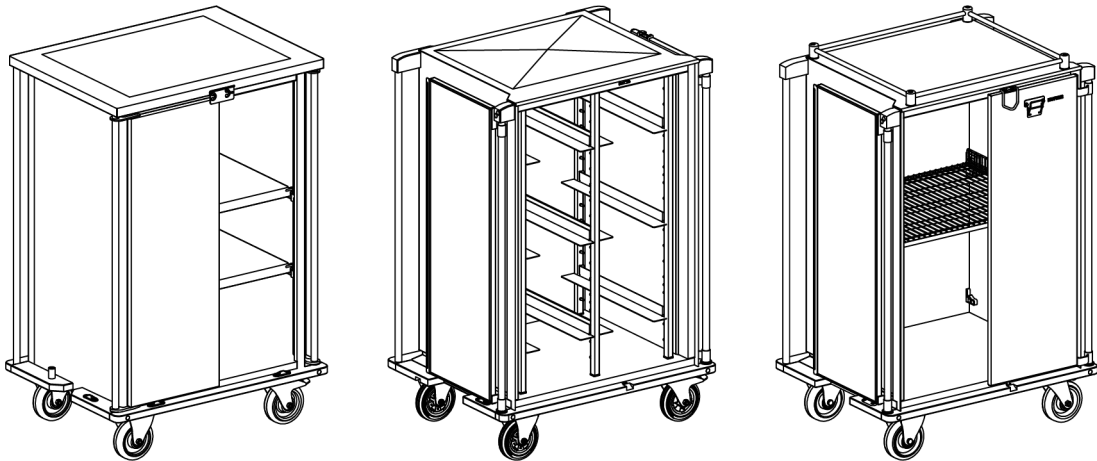
HUPFER

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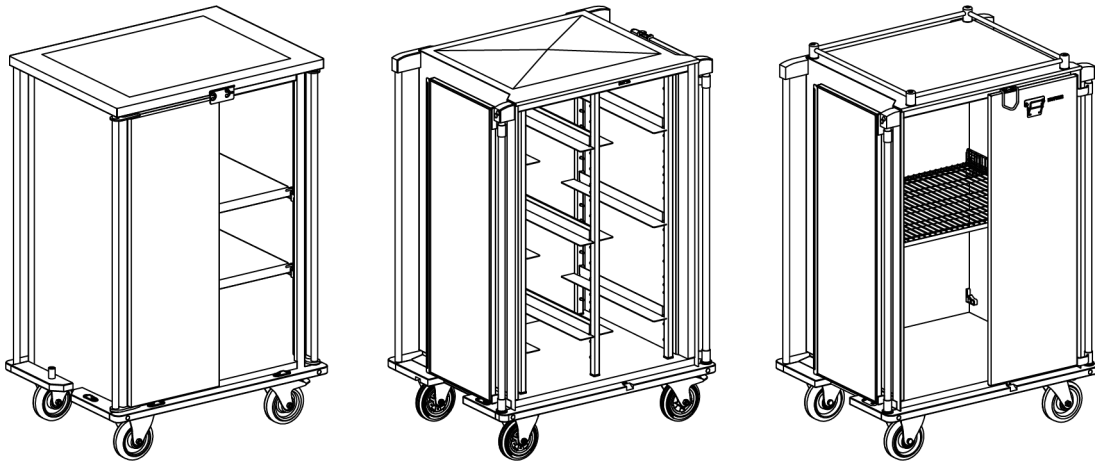
HUPFER

Operating instructions

Sterile goods transport trolleys, container transport trolleys and universal transport trolleys

STW | CTW | ATW

Read the instructions prior to performing any task!

Manufacturer information

Hupfer Metallwerke GmbH & Co. KG

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48653 Coesfeld

Telephone: + 49 (0) 2541 805-0

Email: info@hupfer.de

Internet: www.hupfer.com

Document information

Last update: 26.07.2023

Index: 1, en_GB

All texts, figures and graphic designs are protected by copyright. Reproduction, distribution and issuing are permitted for internal company purposes only. © Hupfer Metallwerke GmbH & Co. KG, Coesfeld 2023

Table of contents

1	General information.....	5
1.1	Product information.....	5
1.2	Target group.....	5
1.3	Symbols.....	5
2	Product description.....	6
2.1	Function.....	6
2.2	Overview.....	7
2.3	Equipment features.....	10
2.4	Variants.....	11
2.5	Equipment options.....	12
2.6	Technical data.....	12
2.7	Rating plate.....	13
3	Safety information.....	14
3.1	Intended use.....	14
3.2	Improper use.....	14
3.3	Safety instructions.....	15
3.3.1	General information.....	15
3.3.2	Transport.....	15
3.3.3	Operation and use.....	16
3.3.4	Cleaning.....	16
3.3.5	Maintenance.....	16
3.3.6	Safety devices.....	16
4	Transport.....	17
4.1	Delivery.....	17
4.2	In-house transport.....	18
4.3	Lifting.....	18
5	Commissioning.....	19
6	Operation.....	20
6.1	Opening and closing doors.....	20
6.2	Moving the cart.....	21
6.3	Loading goods.....	23
6.4	Finishing operation.....	24

7	Care and cleaning	24
7.1	Care instructions.....	24
7.2	Cleaning intervals.....	25
7.3	Permissible cleaning types.....	25
7.4	Permissible cleaning agents.....	27
7.5	Material compatibility.....	28
7.6	Manual cleaning.....	28
7.7	Mechanical cleaning.....	29
7.8	Disinfecting.....	30
8	Maintenance and repair	31
8.1	Maintenance intervals.....	31
8.2	Troubleshooting.....	31
8.3	Repairs and spare parts.....	32
8.4	Recommissioning.....	32
9	Disposal	32

1 General information

1.1 Product information

Product name	Container transport trolleys Sterile goods transport trolley Universal transport trolley
Product type	CTW/S, STW/S CTW/VV, STW/VV ATW/VV

1.2 Target group

These operating instructions are intended for the following groups of people who perform the listed activities with or on the product:

Maintenance engineer

- Maintenance work on mechanical systems or trained activities on the electrical, cooling or heating systems
- Simple repairs
- Appropriately trained employees of the customer or an employee of the manufacturer

Operator

The operator or an authorised person must carry out the work.

- Check the condition of the entire unit
- Determine maintenance requirements

1.3 Symbols

DANGER

“Danger” indicates a hazardous situation that will result directly in death or serious injury.

WARNING

“Warning” indicates a hazardous situation that may result in serious injury.

CAUTION

“Caution” indicates a potentially hazardous situation that may result in minor to moderate injury.

! NOTICE

“Notice” indicates a situation that may result in damage to property.

“Notes” give tips on the correct use of the product.

2 Product description

2.1 Function

Sterile goods and container transport trolleys are intended for the supply and disposal, hygienically safe transport, storage and temporary storage of sterile goods in containers or in baskets.

The universal transport trolley is a unit that can be put into universal use in the field of hospital transport logistics. The universal transport trolley can be used for transporting sterile goods, consumables, pharmacy goods, laundry, food, and waste.

2.2 Overview

Overview of the CTW

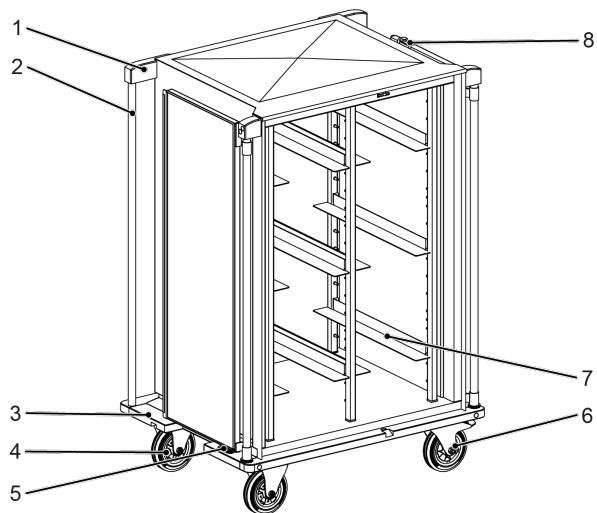
Fig. 1: Overview of the CTW/VV-6

- 1 Impact protection
- 2 Push bar
- 3 Bumper strip
- 4 Fixed castor
- 5 Extension for drain plug
- 6 Swivel castor with lock
- 7 Insert rail
- 8 2-point locking

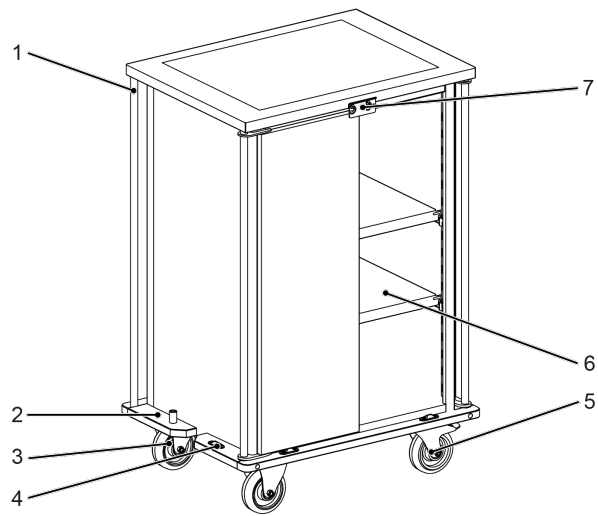
Overview of the STW

Fig. 2: Overview of the STW/S-6

- 1 Transport latch (optional)
- 2 Solid shelf
- 3 Extension for drain plug
- 4 Fixed castor
- 5 Bumper strip
- 6 Swivel castor with lock
- 7 Push bar

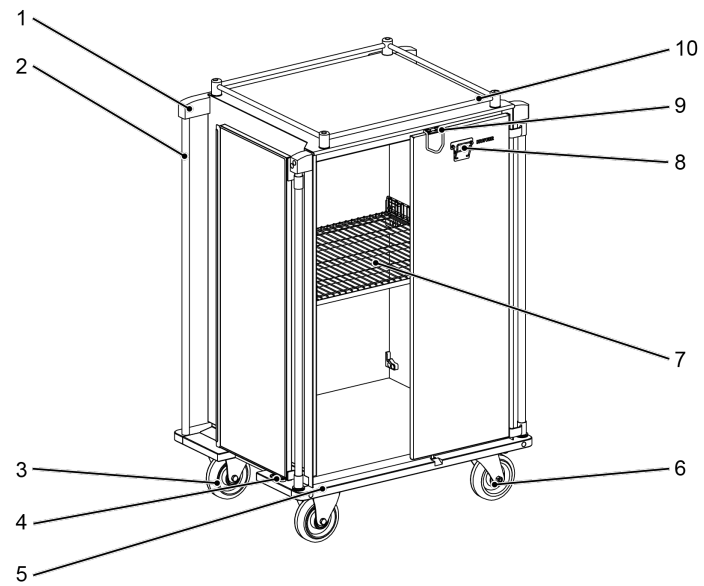
Overview of the ATW/VV

Fig. 3: Overview of the ATW/VV-6

- 1 Impact protection
- 2 Push bar
- 3 Fixed castor
- 4 Extension for drain plug
- 5 Bumper strip
- 6 Swivel castor with lock
- 7 Louvred shelf
- 8 Paper clamp (optional)
- 9 2-point locking
- 10 Top gallery (optional)

2.3 Equipment features

Housing	Made of stainless steel, fully enclosed.
Wheels	Wheels as per DIN 18867-8.
Swivel castors	With locks, actuated locks prevent the castors from swivelling and turning.
Fixed castors	Fixed wheels for easy travel in a straight line.
Push bars	Vertical ergonomic push bar made of round stainless steel tubing on each corner. Simple and ergonomic manoeuvring of the transport trolley with two hands at any handle height.
Doors	<p>With seals to protect the interior from dust and dirt.</p> <p>The transport-secure, 2-point locking device for doors, can be operated with one hand (standard for VV models).</p> <p>An extension for the drain plug keeps the doors closed (standard on S models).</p> <p>The transport latch with sealing flap provides additional protection and stops the doors from opening accidentally (standard on S models).</p>
STW	Two removable solid shelves.
CTW	Height-adjustable, removable U-shape profile rails or L-shape profile rails for holding sterile goods baskets or sterile goods containers.
ATW	Versatile wire shelf that can be hung horizontally or vertically as needed.

2.4 Variants

Model	CTW/S	CTW/VV	STW/S	STW/VV
Construction	Closed	Closed	Closed	Closed
	Single-walled	Single-walled	Single-walled	Single-walled
		Seamlessly welded interior		Seamlessly welded interior
Rail pairs				
Solid shelf	--	--	Standard	Standard
Wire shelf	--	--	Option	Option
Wire shelf with 1/3 extension	--	--	Option	Option
Folding wire shelf	--	--	--	--
L-shape profile rails	Standard	Standard	--	--
StU capacity	3, 6 or 9 insert rail pairs	3, 6 or 9 insert rail pairs	2 shelves	2 shelves
Model				
		ATW/VV	Case carts	
Construction		Double-walled	Closed	
		Seamlessly welded interior	Single-walled	
			STW/S in custom sizes	
Rail pairs				
Solid shelf		--	Standard	
Wire shelf		--	Option	
Wire shelf with 1/3 extension		--	Option	
Folding wire shelf		Standard	--	
L-shape profile rails		--	--	
Capacity		One folding wire shelf; push-in rack with 6 or 9 pairs of insert rail pairs	One shelf for lower version Two shelves for higher version	

Sterile goods or container transport trolleys can be configured in many ways.

2.5 Equipment options

Top gallery	On the trolley top, for securing goods to be transported on the top. The trolley top may be loaded to a maximum of 25 kg.
Paper clamp	For clamping papers and documents.
Card holder	For storage of documents
Shelves	Wire shelf Stainless steel insert frame set for StU and ISO standard containers and baskets.
HGV transport securing device	To protect and fix the transport trolleys securely on an HGV loading platform.
Transport latch	For securing the doors during transport.
Central lock tread	For simultaneous foot actuation of the lock or direction lock of the swivel castors.
Towing device	Docking mechanism and towing device to connect several transport trolleys for traction operation, available as a standard version or as a friction-damped version.

2.6 Technical data

Container transport trolleys

Model		CTW/S-6	CTW/VV-6	CTW/S-9	CTW/VV-9
Dimensions, total weight					
Width ¹	mm	965	965	1,249	1,249
Depth ¹	mm	751	754	751	754
Height ¹	mm	1,469	1,445	1,469	1,440
Weight ¹	kg	100	104	124	133
Payload ¹	kg	100	150	100	150
Rail pairs					
Capacity	--	6 × StU	6 × StU	9 × StU	9 × StU
Insertion type	--	Lengthwise insertion, height adjustable	Lengthwise insertion, height adjustable	Lengthwise insertion, height adjustable	Lengthwise insertion, height adjustable
		6 pairs of insert rails	6 pairs of insert rails	9 pairs of insert rails	9 pairs of insert rails
Chassis	mm	Ø 160	Ø 160	Ø 160	Ø 160
Maximum speed in traction operation	km/h	4	4	4	4

Sterile goods transport trolley

Model		STW/S-6	STW/VV-6	STW/S-9	STW/VV-9
Dimensions, total weight					
Width ¹	mm	965	965	1,249	1,249
Depth ¹	mm	751	754	751	754
Height ¹	mm	1,469	1,445	1,469	1,440
Weight ¹	kg	87	97	99	119
Payload ¹	kg	100	200	150	200
Rail pairs					
Capacity	--	6 × StU	6 × StU	9 × StU	9 × StU
Insertion type	--	Crosswise insertion	Crosswise insertion	Crosswise insertion	Crosswise insertion
Chassis	mm	Ø 160	Ø 160	Ø 160	Ø 160
Maximum speed in traction operation	km/h	4	4	4	4

Universal transport trolley

Model		ATW/VV-6	ATW/VV-9
Dimensions			
Width ¹	mm	754	754
Depth ¹	mm	965	1249
Height ¹	mm	1487	1487
Payload			
Weight ¹	kg	88	114
Payload, total	kg	105	170

¹The figures are approximate values. Deviations are possible.

2.7 Rating plate

- 1 Manufacturer and address
- 2 Type designation
- 3 Order number
- 4 Tare weight
- 5 Payload

3 Safety information

3.1 Intended use

Sterile goods and container transport trolleys are intended for the hygienic and economical distribution, storage, and transport of sterile goods in StU containers or ISO standard containers.

Universal transport trolleys are intended for the economical distribution of sterile goods, supplies, pharmacy goods, laundry, food, and waste.

Items may be transported on transport trolleys with a continuous gallery, provided they are firmly placed and secured against falling down by a non-slip underlay. The trolley top may be loaded to a maximum of 25 kg.

Intended use includes following the prescribed procedures, complying with the given specifications, and using the genuine accessories that are provided or additionally available. Any other use of the equipment is considered inappropriate.

3.2 Improper use

Sterile goods or container transport trolleys and universal transport trolleys may only be moved with the doors closed and on the push bars.

Sterile goods or container transport trolleys and universal transport trolleys without a continuous gallery are not intended for the storage of objects.

They may not be used for transporting living creatures.

Persons must not sit or stand on or in the transport trolley.

In the case of improper use, the manufacturer and suppliers assume no liability for resulting damage. Damage caused by improper use shall result in loss of liability and of guarantee claims.

3.3 Safety instructions

3.3.1 General information

The transport cart may only be used in a technically sound condition, while taking risks and safety into consideration, for its intended purpose and in accordance with the operating instructions.

All operating elements and controls must be in a technically sound condition and functioning reliably.

Before each use, the unit must be checked for any visibly apparent damage and defects. If damage occurs, immediately notify those responsible and take the unit out of use.

Changes or conversions are only permitted after consultation and written consent from the manufacturer.

Transport trolleys can move off by themselves and in an uncontrolled manner if the wheels are not locked.

Transport trolleys can move off by themselves and in an uncontrolled manner if the wheels are not locked and the direction lock is engaged.

3.3.2 Transport

If the transport cart needs to be lifted, only use lifting gear and load-bearing equipment approved for the total weight of the transport cart.

Only use transport vehicles that are approved for the total weight of the transport trolley.

The transport trolley does not have any attachment points for eyelets or similar lifting aids.

Transport trolleys on which the direction lock has been released roll from side to side when in motion. Transport trolleys on which the direction lock has been released can move off by themselves and in an uncontrolled manner on inclined surfaces, e.g. loading ramps. Only release the direction lock briefly when the transport trolley is at a standstill and on level ground, e.g. to move it sideways.

If the transport trolley is transported, it must be additionally secured during transport. The locks on the castors are not sufficient as transport securing devices. They must not be braked during transport on an HGV because the locking mechanism will be damaged by the forces that occur.

3.3.3 Operation and use

Transport trolleys can move off by themselves and in an uncontrolled manner if the wheels are not locked. Secure the stationary transport trolley against rolling away.

Do not park on sloping ground. Only move transport trolleys with the locks released. Moving with the locks engaged will damage the chassis. The locks are not brakes. Only engage the locks when the transport trolley is stationary.

Keep an overview of the route and move the transport trolley with foresight, for example by pulling when necessary. Watch out for children, other persons and obstacles. Do not move the transport trolley over steps. Two people should move the transport trolley on slopes.

Only move loaded transport trolleys with the doors closed to prevent goods to be transported from falling out.

Never try to catch a transport trolley that is falling over. Transport trolleys are heavy and can cause serious injury. Think carefully before starting to move the transport trolley.

Always push the container in completely. There is a risk of tipping if the container is not fully inserted.

3.3.4 Cleaning

When cleaning in the transport trolley washing device, ensure that the interior is completely emptied and that all movable parts are adequately secured.

3.3.5 Maintenance

Troubleshooting may only be carried out by maintenance technicians.

3.3.6 Safety devices

Locks on the wheels. The wheel locks must be applied when the transport trolley is stationary to prevent it from rolling away.

4 Transport

WARNING

Risk of crushing

Improper use or incorrect means of transport may cause the transport trolley to tip over, slip or fall. If not adequately secured, there is a danger of serious crushing injury and damage to the transport trolley.

- Secure each transport trolley with appropriate transport securing devices during transport.
 - Fix the wheels in the “straight ahead” position.
-

NOTICE

Damage to the locks

The forces that occur during transport by lorry cause damage to engaged locks.

- Secure the transport cart with a transport securing device.
 - Release the locks during transport by lorry.
-

HGV transport securing device

The equipment option “HGV transport securing device” is available for securing the transport trolley during HGV transport.

4.1 Delivery

Delivery is performed by a transport company, whose specialists are responsible for securing the load during transport. At the place of delivery, the transport cart can then be moved on its own wheels.

4.2 In-house transport

WARNING

Risk of crushing

A tray transport trolley rolling in an uncontrolled way may crush people.

- Never leave the transport cart to roll unattended.
- Only drive uphill or downhill if you can control the transport trolley. If necessary, involve a second person.
- Only park the transport trolley on level surfaces.
- Each time you park the transport trolley, secure it with both locks against rolling away.
- Fix the wheels in the “straight ahead” position.

Only use transport vehicles that are approved for the total weight of the equipment.

1.  Close the doors.
2.  Release the lock.
3.  Fix the wheels in the “straight ahead” position.

4.3 Lifting

The transport cart does not have any attachment points for eyelets or similar lifting aids.

Always transport the transport cart upright if possible. It can be lifted using a lifting platform, for example.

5 Commissioning

WARNING

Risk of crushing

A tray transport trolley rolling in an uncontrolled way may crush people.

- Never leave the transport cart to roll unattended.
- Only drive uphill or downhill if you can control the transport trolley. If necessary, involve a second person.
- Only park the transport trolley on level surfaces.
- Each time you park the transport trolley, secure it with both locks against rolling away.
- Fix the wheels in the “straight ahead” position.

NOTICE

Instructing operating staff

Before commissioning, operating staff must be instructed how to operate the transport cart correctly.

Help protect our environment. Separate the packaging materials and recycle them. Dispose of:

- Packaging and foils in the (yellow) recycling bin.
- Cardboard boxes and paper in the paper recycling bin.

Clean the unit thoroughly before first use (see cleaning instructions).

➔ Check the functions of the equipment:

- Functionality of all mechanical parts.
- Functionality of the controls.
- Functionality of the lock.
- Functionality of the direction lock.

6 Operation

CAUTION

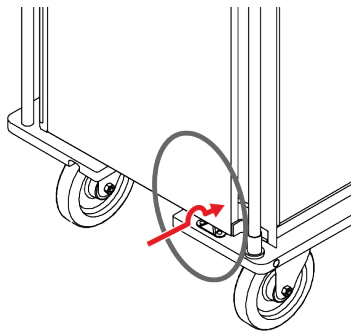
Contamination of sterile goods

Contamination of sterilised goods represents a health hazard.

- Observe the operator's instructions for maintaining sterility.

6.1 Opening and closing doors

Extension for drain plug


To lock the open doors in place, an extension for the drain plug is attached on the lower bumper strips.

1. → Open the door.
2. → Flip the door over until the door snaps into place behind the extension for the drain plug.

Transport latch

The transport latch provides additional protection for the doors to stop them unintentionally springing open, for example during HGV transport.

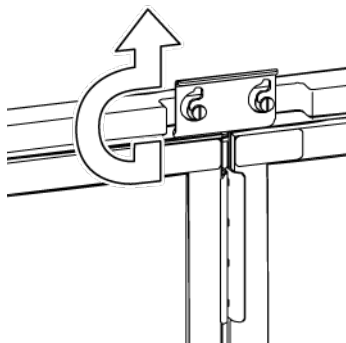You can open the transport latch.

- Open the transport latch as shown in the figure.
 - ➔ You can open the doors at the door lock.

You can close the transport latch.

- Close the transport latch by reversing the steps above.
 - ➔ The doors are additionally locked.

6.2 Moving the cart

WARNING

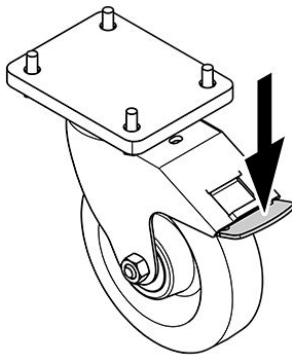
Risk of crushing

A tray transport trolley rolling in an uncontrolled way may crush people.

- Never leave the transport cart to roll unattended.
- Only drive uphill or downhill if you can control the transport trolley. If necessary, involve a second person.
- Only park the transport trolley on level surfaces.
- Each time you park the transport trolley, secure it with both locks against rolling away.
- Fix the wheels in the "straight ahead" position.

Engaging the locks


Swivel castors with locks can block the wheel from turning and swiveling. When the locks on both swivel castors are engaged, the transport cart is secured against rolling away.

→ Engage the lock as shown on the left.

→ The wheel cannot turn or swivel.

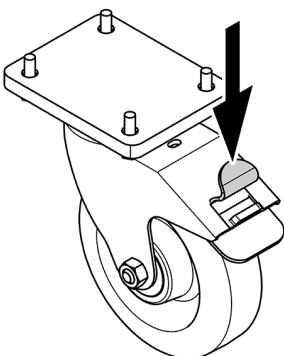Releasing the lock

→ Release the lock as shown on the left.

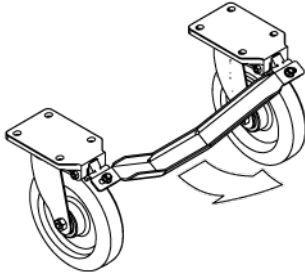
→ The wheel can turn and swivel.

Actuating the central lock tread

Optionally, the transport carts have a central lock tread. With the central lock tread, the locks on both swivel castors are operated simultaneously. The central lock tread for the locks is marked *[Stop.]*

→ Engage the lock as shown on the left.


➔ The wheel cannot turn or swivel.

Releasing the central lock tread

→ Release the lock as shown on the left.

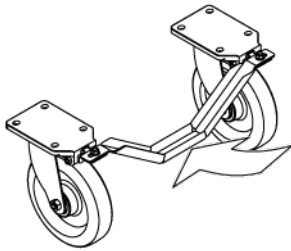➔ The wheels can turn and swivel.

Actuating the direction lock

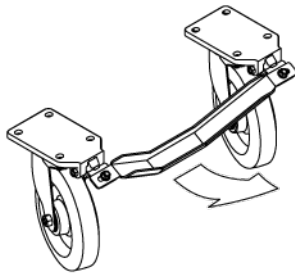
WARNING

Contusion hazard

Transport carts on which the direction lock has been released roll from side to side when in motion. Transport carts on which the direction lock has been released can move off under their own weight and in an uncontrolled manner on inclined surfaces, e.g. loading ramps. The transport cart is heavy, and if it rolls away out of control, there is a risk that people may be injured by being hit or crushed.

- Activate the direction lock for the wheels.
- Only release the direction lock when the transport cart is at a standstill and on level ground.

Optionally, the transport carts have a central lock tread. With the central lock tread, the wheels are fixed straight ahead.


- └ Lock the direction of the wheels as shown on the left.
- ➔ When the direction of the wheels is fixed, the cart is more stable when travelling on longer routes.
- ➔ The direction lock snaps in audibly.

Release the direction lock

Only release the direction lock to manoeuvre in confined spaces, e.g. to move the appliance sideways. Afterwards, reactivate the direction lock.

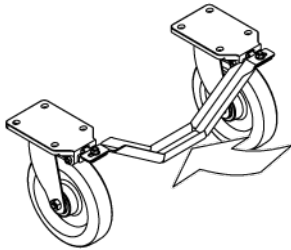- └ Release the direction of the wheels as shown on the left.
- ➔ The wheels can move freely again.

6.3 Loading goods

Container transport trolley and sterile goods transport trolley

1. └ Secure the transport trolley against rolling away with the locks.
2. └ Slide the containers into the side guides or place them on the shelves.
3. └ Close the doors.
4. └ Lock the transport trolley with the transport securing device.
5. └ Release the lock.
6. └ Push the transport trolley to the destination.

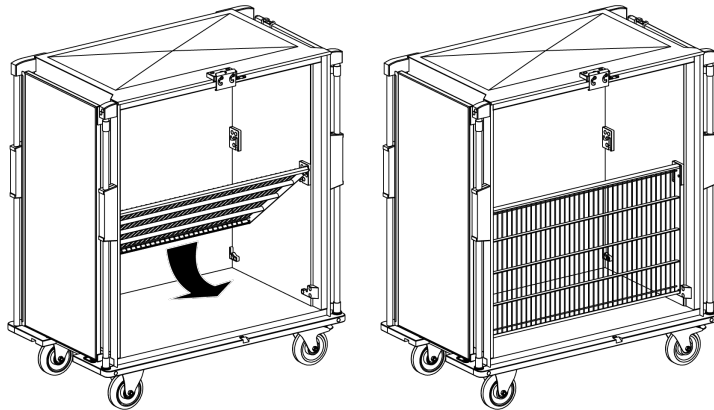
Preparing the universal transport trolley

The folding wire shelf can be used horizontally as well as vertically.

When the wire shelf is inserted horizontally, it can be used for the transport and distribution of supplies and consumer goods.

When the wire shelf is inserted vertically, it can be used for the transport and disposal of laundry or even waste.

Depending on the intended use, the universal transport trolley must be converted before use.

- > Release the shelving bolts of the wire shelf from the holders and fold them forward.
 - ➔ The universal transport trolley is converted when the shelving bolts of the wire shelf fully engage in the holders.

6.4 Finishing operation

1. —> Secure the transport trolley with the locks when stationary.
2. —> Remove all loose parts.
 - ➔ The transport trolley is ready for cleaning.

7 Care and cleaning

7.1 Care instructions

Door seals

The door seals become brittle over time, for example due to high temperature fluctuations.

- > Treat them at regular intervals with commercially available care products for lip seals.

7.2 Cleaning intervals

Interval	Maintenance work
After each use and according to the in-house hygiene concept	Cleaning the transport trolley
As needed and according to the in-house hygiene concept	Disinfecting the transport trolleys

7.3 Permissible cleaning types

CAUTION

Spreading water

Wet and dripping transport trolleys distribute water on paths. There is a severe risk of slipping on wet paths.

- Dry the transport trolley after cleaning.
- Remove the water from the interior, e.g. by tilting the trolleys 5°.

NOTICE

Deformation at high temperatures

The polymers used deform at high temperatures.

- Restrict the temperature to max. 120 °C.

NOTICE

Bearing damage

The wheel bearings are greased. Washing out the bearing grease will result in bearing damage.

- Do not direct jets directly at the wheel bearings.

NOTICE

Damage due to high-pressure cleaning

Lip seals on the door and housing can become loose due to direct high-pressure jets and damage will result.

- Do not point the spray directly at the seals.
- Keep a distance of at least 30 cm between the nozzle and the device.
- Use a flat stream nozzle.

Cleaning type	Definition of the cleaning operation	Approval for			
		CTW/S STW/S	CTW/VV STW/VV	ATW/VV	
Manual cleaning	Dry	Cleaning with dry cleaning agents			
	Moist	Cleaning with a damp cloth and degreasing liquid cleaner			
	Wet	Cleaning with a wet sponge and degreasing liquid cleaner			
Water hose	Pressure ≤ 8 bar	Cleaning with a spray jet and degreasing cleaning agents if required. <ul style="list-style-type: none">Do not spray into bearings.			
High-pressure/steam cleaning	High pressure ≤ 150 bar	Cleaning the surfaces: <ul style="list-style-type: none">max. 30-90 secondsDo not spray into bearingsUse a flat stream nozzleKeep a distance of at least 30 cm			
	Wet steam $\leq 135^\circ$ C	Cleaning the surfaces: <ul style="list-style-type: none">max. 30-90 secondsShort exposure time for individual sectionsDo not spray into bearings			
Transport trolley washing devices	Water temperature $\leq 90^\circ$ C	Spraying with cleaning and rinsing agent: <ul style="list-style-type: none">max. 30-90 seconds			

 The CTW/S and STW/S transport trolleys have undercuts and cannot be cleaned when cleaned in washing devices.

7.4 Permissible cleaning agents

! NOTICE

Incorrect cleaning agents

Incorrect treatment of the installed material surfaces will result in damage and corrosion.

- Only use cleaning agents approved for cleaning.
- Follow the instructions of the cleaning agent manufacturer.

Suitable cleaning agents

The manufacturer cannot assess the suitability of specific detergents or disinfectants.

- Check the notes on the cleaning agents or disinfectants to find out whether they are suitable for the materials of the transport cart.
- If in doubt, ask the manufacturer of the agent if it is suitable for the materials of the transport cart.



Cleaning agent	Approval
Degreasing agent	✓
■ Neutral liquid cleaner	
Salty cleaning agents	✗
■ Table salt	
■ Hydrochloric acid	
Sulphuric cleaning agents	✗
■ Sulphur dioxide	
■ Sulphuric acid	
Chlorinated cleaning agents	✗
■ Chlorine cleaner	
■ Sea water	
■ Bleach	
Ferrous material	✗
■ Water with iron content	
Low-oxygen cleaning agents	✗
■ Low-oxygen water	
■ CO2 atmosphere	
Neutral cleaning agents	✓
■ Neutral cleaning agents pH 7	
■ Slightly acidic cleaning agents pH 6 – 7	
■ Slightly alkaline cleaning agents pH 7 – 8	
Acidic cleaning agents pH 1 – 5	✗
■ Acids	

Cleaning agent	Approval
Alkaline cleaning agents pH 8 – 14	
■ Alkalis	
Abrasive cleaning agents	
■ Scouring pad	
■ Pot scrubber	
Non-abrasive cleaning agents	
■ Uncoated sponges	
■ Soft cloths	

7.5 Material compatibility

Material	Components
Stainless steel	Housing, doors, interior, lid
PE 500 (polyethylene)	Bumper strip
PA (polyamide)	Extensions
Silicone	Seals

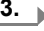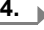7.6 Manual cleaning

3.  Clean the outer surfaces of the transport cart after use or as required.
4.  For wet cleaning, use a soft cloth or an uncoated sponge.
5.  Use degreasing liquid cleaners.
6.  Rinse so that all detergents and disinfectants are completely removed.
7.  Dry the interior and exterior surfaces after cleaning.

7.7 Mechanical cleaning

CAUTION

Inadequate cleaning

The hygienic cleaning of the transport trolley is determined by the settings of the washing device. Incorrect settings can lead to inadequate cleaning.

- Check the hygienic condition of the transport trolley after cleaning.
- Correct the settings of the washing device if necessary.

NOTICE

Damage caused by washing devices

The operator must ensure that the washing device and the transport trolley are compatible. If transport trolleys are cleaned in incompatible washing devices, there is a risk of damage to the transport trolley as well as the washing device.

- Before machine cleaning, check the compatibility of the transport trolleys with the washing device.
- Observe the instructions from the washing device manufacturer.
- Adjust the washing device to the dimensions of the transport trolley.

NOTICE

Risk of corrosion

If residues of cleaning agents and disinfectants are not completely removed by thorough rinsing, the salts in the residue may cause corrosion to form on the stainless steel surfaces of the transport trolley.

Transport trolleys are approved for mechanical cleaning in washing devices if the washing device meets the requirements stipulated in the "Approved cleaning types" chapter. If the operating conditions of the washing device used deviate from the specifications in the "Approved cleaning types" chapter, the manufacturer's warranty obligation will become void.

Please observe the following when cleaning in washing devices:

- The instructions of the washing device manufacturer
- The "Special care instructions for stainless steel" in the manufacturer's download area.
- The instructions of the cleaning and rinsing agent manufacturer
- Only wash technically faultless transport trolleys in washing devices.
- At the latest during the drying phase, the transport trolley must be tilted 5 ° to the door side to ensure that water can drain off optimally. The transport trolley may be tilted to a maximum of 10 °.

The washing performance of the washing device is affected by the duration, temperature as well as mechanical (water pressure) and chemical (cleaning agent and rinsing agent) factors. Optimise these parameters to improve the cleaning result.

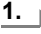
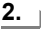
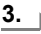
Before washing

! NOTICE





Appliance damage

On washing devices with rotating platforms, the doors can pop out of the lock due to the rotating shafts. The uncontrolled movement can cause damage to the doors and washing device.

- Additionally secure the doors.
-

1.  Open the doors fully.
 - ➔ The doors snap into place behind the extension for drain plug.
2.  Additionally secure the doors when the trolley is being cleaned in washing devices with rotating platforms.
3.  Empty the transport trolley completely, including the top gallery.
 - ➔ The transport trolley is ready for washing.

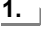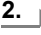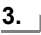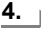After washing

1.  The transport trolley must be free of cleaning agents and disinfectants.
2.  Inspect the transport trolley for damage to the housing, trolley doors, castors and locks from the washing process. Only use transport trolleys that are technically faultless.
3.  Allow the trolley to cool and dry with the doors open for 10–15 minutes.
4.  Dry the transport trolley completely. Leaking water creates a slip hazard.
 - ➔ The transport trolley is ready for operation.

7.8 Disinfecting

All the surfaces can be disinfected with commercially available surface disinfectants.

! NOTICE

Damage to property

High concentrations of disinfectant can cause corrosion damage.

- Follow the instructions of the disinfectant manufacturer.
-

8 Maintenance and repair

WARNING

Unqualified staff

Danger from incorrect handling of mechanical components.

- Only qualified personnel may carry out maintenance work.

8.1 Maintenance intervals

Depending on the load, the transport trolley should be checked regularly for to ensure it functions properly. The manufacturer recommends the following components are checked and maintained at regular intervals.

Interval	Maintenance work	Personnel
daily	Carry out a visual inspection of the entire unit.	Operator
annually	<ul style="list-style-type: none"> ■ Check entire unit for distortion and mechanical damage ■ Check effectiveness of the locks ■ Check mechanical strength of the wheels ■ Check functionality of the central lock treads 	Maintenance engineer

8.2 Troubleshooting

In the event of malfunctions and complaints within the guarantee period, contact our service partners. After the guarantee period has expired, have any necessary repair work carried out by our service partners or electricians.

Fault description	Cause	Remedy	Personnel
Transport trolley pulls to one side during transport	Wheel bearings damaged	Replace worn wheels	Maintenance engineer
Running noises made by wheels	Wheel bearings damaged	Replace worn wheels	Maintenance engineer
Castor lock no longer working	Lock worn out	Replace lock	Maintenance engineer
Doors do not close correctly	Door seal defective	Replace door seal	Maintenance engineer
	2-point locking tension spring defective	Replace tension spring	Maintenance engineer

8.3 Repairs and spare parts

Spare parts and accessories are available from our customer service. Replace defective components with genuine spare parts. Only then can safe and reliable operation be guaranteed. When ordering spare parts or requesting customer service, always state the order number and the information on the rating plate. This information helps avoid follow-up queries and speeds up the process.

8.4 Recommissioning

The transport cart may only be used in a technically sound condition, for its intended use, and in accordance with the operating instructions. Check the unit for visible damage before each use. In the event of damage, inform the responsible departments immediately. Only operate units that are in faultless technical order.

- Check the transport cart functions, especially the lock.
- Clean the unit thoroughly before putting it back into operation.

9 Disposal

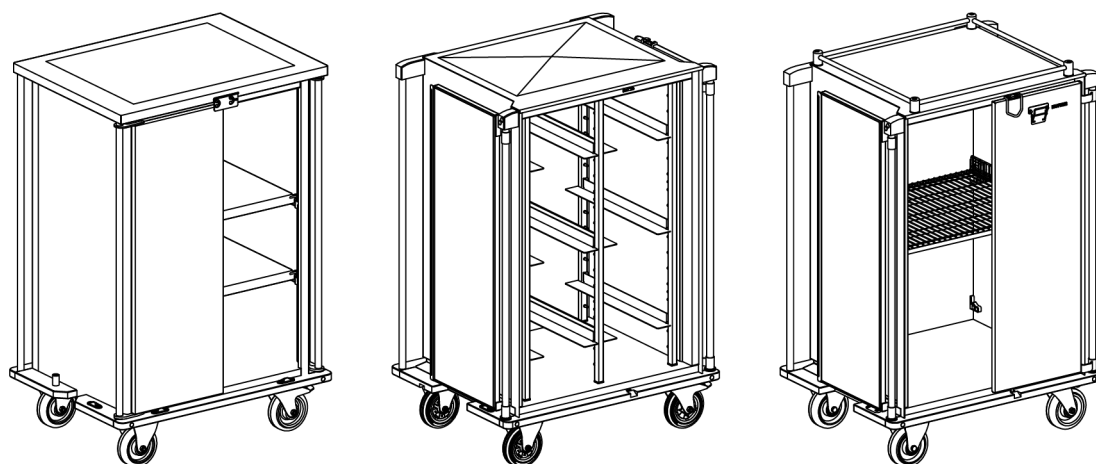
Help protect our environment. Make sure that the materials are recycled.

- Have the unit disassembled and disposed of by recycling companies. All the materials are recyclable.

➔ Take the components to a designated disposal centre.

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Tutti i testi, le illustrazioni e i disegni grafici sono protetti da copyright. La riproduzione, la distribuzione e l'esposizione sono consentite esclusivamente per scopi aziendali. © Hupfer Metallwerke GmbH & Co. KG, Coesfeld 2023